

AESCULAP®

Instructions for Disposable Cables

CAUTION: Federal (USA) law restricts this device to sale by or on the order of a physician

This product is disposable, supplied sterile and intended for single use only. Prior to the expiration date, contents are sterile if the package has not been opened or damaged. Do not resterilize.

INDICATIONS: Disposable cables are designed to connect an electrosurgical device to an electrosurgical generator. Please refer to the labeling to determine the particular connectors and specific use of the cable.

CONTRAINDICATIONS: Any use of this instrument for tasks other than for which it is indicated, will usually result in a damaged or broken instrument.

SETUP and USE

WARNING: CONNECT BIPOLAR ACCESSORIES TO THE BIPOLAR RECEPTACLE AND MONOPOLAR ACCESSORIES TO THE MONOPOLAR RECEPTACLE ONLY. IMPROPER CONNECTION OF ACCESSORIES MAY RESULT IN INADVERTENT ACCESSORY ACTIVATION OR OTHER POTENTIALLY HAZARDOUS CONDITIONS.

Attach the sterile cable to the sterile instrument, ensuring that the contact pins are fully seated in the cable receptacles. Connect the cable to the generator only while it is in the "OFF" or "STANDBY" mode. Failure to do so may result in injury or electrical shock to the patient or operating room personnel.

At the lowest power setting, test the instrument connections by pressing the generator's activating switch. If the generator fails to activate, check the connection with the cable.

CUSTOMER RETURNS / COMPLAINTS

Aesculap, Inc

615 Lambert Pointe Drive
Hazelwood, MO 63042
Phone: 1.888.237.2852 ext. 5938
Email: Aesculap_QAComplaints@BBMUS_Service@BBMUS@BBRAUN.com
www.aesculapusa.com

Distributed by:

AESCULAP

3773 Corporate Parkway
Center Valley, PA 18034

SOP-AIC-5001188 Rev. 01 07/13



SEE INSTRUCTIONS FOR USE

LOT

BATCH CODE

Manufactured by:
Kirwan Surgical Products, Inc.
180 Enterprise Drive
Marshfield, MA 02050, USA
Tel: 1-888-547-9267
Made in USA

REF

CATALOG NUMBER



USE BY:
YEAR-MONTH

STERILE R

STERILE - STERILIZED BY IRRADIATION

Sterility guaranteed unless package opened or damaged. Do not resterilize.

Rx Only

CAUTION: Federal (USA) law restricts this device to sale by or on the order of a physician.



SINGLE PATIENT USE