Instructions for Use

Bard-Parker® Protected Disposable Scalpel Important Usage Guidelines Non-Sterile

Intended Use

Intended for tissue separation and other procedures that require a sharp surgical blade to puncture or cut.

Precautions

- This device is single use only. Reuse of this device may lead to patient cross contamination and/or device failure.
- Ruler shown on scalpel handle should be used as an indicator only, and not for accurate measurements.
- During use avoid twisting, bending or putting excessive force or strain on the blade in order to help prevent breakage.
- A singular blade is intended to cut tissue up to 24 inches.
- If blade becomes dull or breaks, dispose and replace product.
- If safety shield or housing malfunction, dispose and replace product.

Sterilization Instructions

1. Place device into a kit or sterilization tray.
2. Using Ethylene Oxide sterilization method, sterilize according to the following parameters

Cycle Parameter	Specification	Duration
Preconditioning		
Temperature	110°F +/- 10°F	
Humidity	65% +/- 15%	10.35 Hrs -72 Hrs
Chamber		
Vacuum	21" +/- 1.0 inHgA	
Humidification/Dwell	22.5" +/- 1.0 HgA	60 +30/-0 minutes
Gas Inject/Dwell	10.3" +/- 0.5 Hg rise	360 +5/-0 minutes
Jacket Temperature	135°F	
Aeration		
Temperature	120°F	24 Hrs – 72 Hrs Max

To Use

1. Depress top button on protective shield.
2. Retract shield to rear locked position to fully uncover blade.

To Cover

1. Depress top button on protective shield.
2. Advance shield to forward locked position at lock indicator line.

Disposal

Dispose protected scalpel in an approved sharps container in accordance with your facility's sharps disposal procedures.

Does not contain natural rubber latex

Rx ONLY Caution: Federal law restricts this device to sale by or on the order of a physician

Aspen Surgical Products, Inc.
6945 Southbelt Dr. SE
Caledonia, MI 49316 USA
Phone 616-698-7100
Toll-Free 888-364-7004
www.aspensurgical.com

EC REP

Hill-Rom Limited
Clinitron House, Ashby Park
Ashby De La Zouch
Leicestershire, LE65 1JG
England

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