

----- TEAR TO OPEN -----

BD Alaris™ Pump Infusion Set

Back Check Valve

2 SmartSite™ Y-sites

REF 2420-0500

≈ 25 ml; ≈ 297 cm (117 in)

WARNING:


TO PREVENT FREE-FLOW, CLOSE SET CLAMP WHEN SAFETY CLAMP ON PUMP SEGMENT IS OPEN.

Directions: Use Aseptic Technique

1. Prior to every access, swab y-site with 70% isopropyl alcohol and allow to dry (approximately 30 seconds).
2. Close roller clamp.
3. Spike solution container and hang.
4. Squeeze and release middle of drip chamber until approximately 2/3 full. Open vent cap on spike after filling drip chamber if container requires venting.
5. Open roller clamp to prime. If applicable, attach syringe to y-site and aspirate miniscule air bubbles. Close roller clamp.
6. Insert pump segment into pump. (For proper placement information refer to pump User Manual.)

Cautions and Recommendations:

- Do not place on sterile field. Do not use if protective end caps are not in place, if package is open, or if product is damaged or leaks.
- Safety clamp on pump segment is closed when white slide clamp is extended. To open, press grooved blue tab while pushing the white slide clamp into the blue housing.
- If y-site is accessed by a needle, the valve will be damaged causing leakage. Replace set immediately.
- Do not use blunt cannula with y-site.
- Do not leave slip luer syringes unattended in y-site.
- During use of y-site, fluid may be observed between the housing and blue piston. This fluid does not enter the fluid path and requires no action.
- For product questions, contact your BD representative. Consult facility protocols. The CDC, Infusion Nurses Society, and other organizations publish information useful in developing facility guidelines.
- Set can be used for pump or gravity infusion.
- The set should be changed according to facility protocol or in accordance with currently recognized guidelines for IV therapy.
- To dispose of this set adhere to local and/or other governing regulations for medical device and/or biohazardous waste disposal.

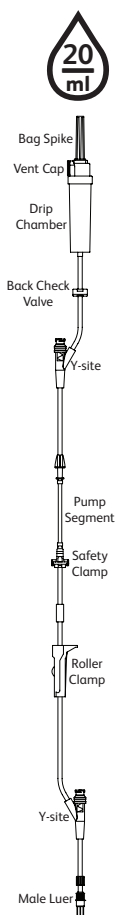

Symbols Glossary

<http://www.bd.com/symbols-glossary>

Do not reuse	Contains or presence of phthalate: DEHP	Keep away from sunlight	Consult instructions for use
Do not use if package is damaged	Fluid path sterilized using irradiation	Fragile, handle with care	Caution
Do not resterilize	Catalogue number	Temperature limit	Manufacturer
Nonpyrogenic	Batch code	Keep dry	Use by date
Drops per millilitre			

Rx Only

Natural Rubber Latex is not part of the material formulation.

BD Switzerland Sàrl, Route de Crassier 17, Business Park Terre-Bonne, Batiment A4, 1262 Eysins Switzerland
 Distributed by CareFusion San Diego, CA USA 1.800.854.7128
 Made in Mexico
 630-01393 Rev 00
 © 2017 BD. BD, the BD Logo and all other trademarks are property of Becton, Dickinson and Company.

LOT

