
BD Insyte™ Autoguard™

Shielded IV Catheters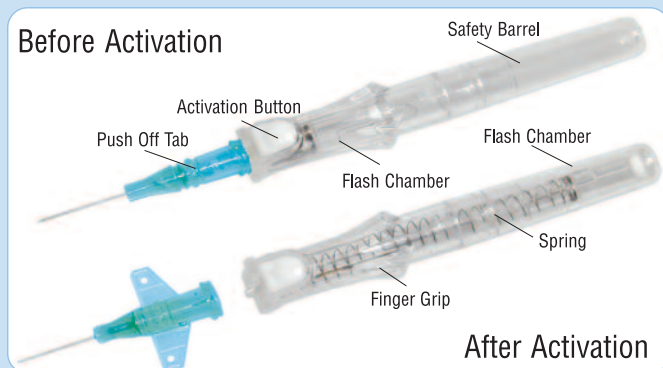

POINTS TO PRACTICE

1 PREPARATION

- Make sure all items are accessible throughout the procedure
- Prepare site according to your facility's policy and procedure
- Prior to venipuncture hold catheter hub and **rotate the barrel 360 degrees**
- Make sure catheter is seated back in the notch

2 VENIPUNCTURE

- Approach vein slowly at a low angle
- Observe early flashback along the catheter (20, 22, 24 gauge only) INSTAFLASH
- In larger gauge sizes observe flash behind white button

3 ADVANCEMENT

- On flashback visualization, lower catheter almost parallel to the skin
- Advance *entire unit* slightly before threading catheter
- Thread catheter into vein while maintaining skin traction

4 NEEDLE REMOVAL

Before Pressing the Activation Button

- Release tourniquet
- Apply digital pressure beyond the catheter tip
- Gently stabilize catheter hub
- Press the white button





5 SECUREMENT

- Secure catheter and apply sterile dressing according to your facility's policy and procedure

CAUTION REMINDERS

- **Do Not** withdraw needle from catheter hub before pressing white button.
- **Needle should be retracted prior to disposal** in a puncture-resistant, leak-proof sharps container.
- **Never Reinsert Needle** into the catheter as this could shear the catheter.
- **Do Not Use Scissors** at or near the insertion site.

Refer to package insert for complete instructions for use and safety information.

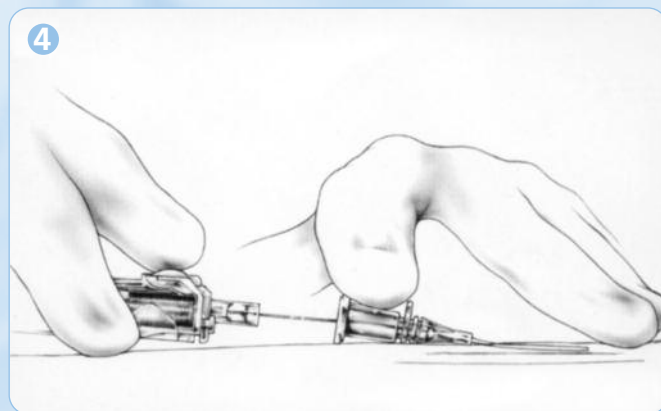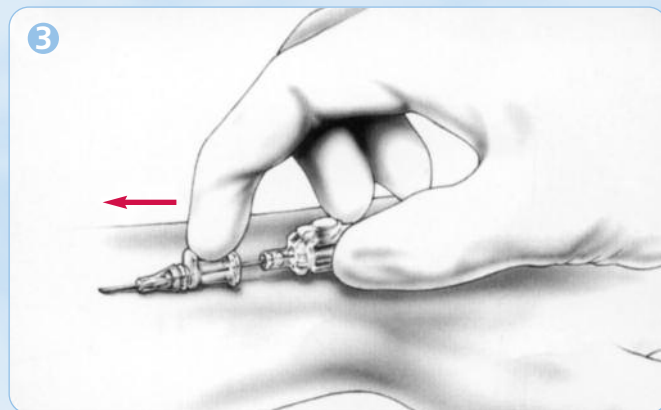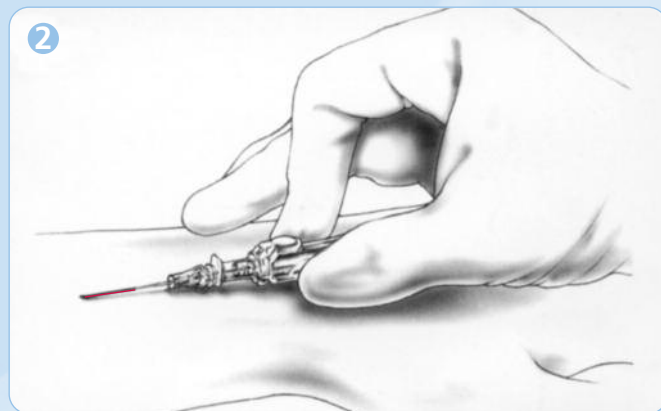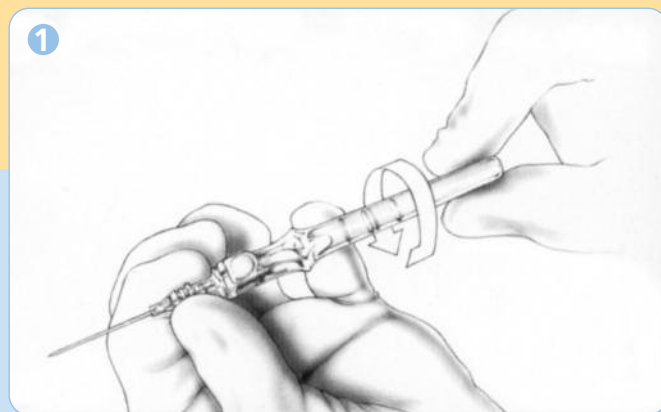BD Medical
2100 Derry Road West, Suite 100
Mississauga, Ontario L5N 0B3
1.800.268.5430

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