


BD medication preparation and administration with BD SafetyGlide™ Needle

Important usage guidelines

Preparation: BD™ Blunt Fill and BD™ Blunt Filter Needle



- 1 Attachment**
- Attach the BD™ Blunt Fill or Filter Needle to any standard luer-lock hypodermic syringe. Twist until firmly seated. Pull the red needle shield straight off.

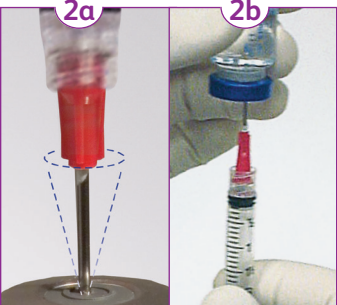

- 2 Aspiration**
- 2a.** Puncture stopper at 90° with the BD™ Blunt Fill Needle to minimize risk of catching internal wall of vial stopper. Note: For glass ampoules use the BD™ Blunt Filter Needle.
- 2b.** Draw up medication in accordance with your institution's protocol.



- 3 Recapping**
- Passively recap BD™ Blunt Fill or Blunt Filter Needle.




- 4 Disposal**
- Detach needle from syringe and immediately discard into an approved sharps collector after single use.

Remember

- The BD™ Blunt Fill and Blunt Filter Needles are not to be used for skin injections.
- Follow your institution's protocol for aseptic technique.
- Follow your institution's policy for safe disposal of all medical waste.
- The BD™ Blunt Filter Needle is to be used with glass ampoules only.
- The BD™ Blunt Filter Needle is a one-way filter and should be removed once fluid is filtered into the syringe.
- To connect the BD™ Blunt Fill or Blunt Filter Needle to a luer-slip syringe, push firmly.

Administration: BD SafetyGlide™ Needle

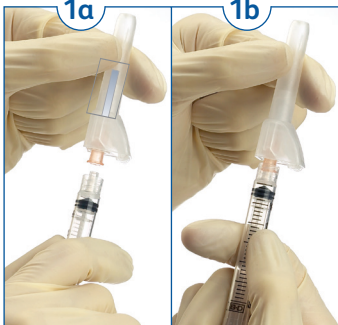

- 1 Attachment**
- 1a.** Attach the BD SafetyGlide™ Needle to any standard luer lock or luer slip hypodermic syringe. Twist until firmly seated to connect to a luer lock syringe.
- 1b.** Pull shield straight off needle to avoid damaging needle point.



- 2 Injection**
- Administer medication in accordance with your institution's protocol. For user convenience, the needle's "bevel up" position is oriented to the safety cover.



- 3 Activation**
- Activate safety mechanism immediately after removal from patient by pushing the lever arm completely forward until needle tip is completely covered.
- Visually confirm that the lever arm has fully advanced and the needle tip is covered. Use one-handed technique and activate away from self and others. After activation or if unable to activate, discard immediately in an approved sharps collector after single use.

Remember

- To connect the BD SafetyGlide™ Needle to a luer slip syringe, push firmly.
- Follow your institution's protocol for aseptic technique and skin preparation.
- Follow your institution's policy for safe disposal of all medical waste.

BD™ Blunt Fill Needle



BD™ Blunt Filter Needle



Note hub color difference between blunt fill vs blunt filter needle