



Medication preparation

Medication delivery



Sharps disposal



Cardinal Health™ Monoject™ Magellan™ Hypodermic Safety Needles

Instructions for use

1



Select the appropriate Monoject™ Magellan™ Hypodermic Safety Needle for your procedure. All Monoject™ Magellan™ Safety Needles are ISO color-coded.

2



Open package using an aseptic technique by peeling back paper tab, then remove safety needle.

3



Secure the needle by holding the needle and twisting the syringe clockwise to fully seat the needle. Remove protective needle sheath.

4



Draw up medication and administer injection using an aseptic technique in accordance with your facility's protocol.

Lock the safety shield using any of the following activation methods:

5



Thumb



Finger



Flat surface

6



Verify locked position through audible and tactile "click." Locked position will completely cover needle tip.

7



Once safety shield is locked, immediately dispose of needle and syringe in an approved sharps container.

For more information, visit cardinalhealth.com/monoject
To place an order, log on to The Cardinal Health MarketSM or
call customer service at **1.800.964.5227**