

TDE015500

STERILE R



Rx only

List No. 6543-01
Sterile Pack

T-U-R Y-Set

Nonvented, 96 Inch

For Transurethral Resection Procedures
For Flexible Irrigation Container Systems

Natural rubber latex has not been used in the manufacture of the device. Contents sterile and nonpyrogenic in intact unit package. Use aseptic technique. Remove caps when required and secure connections.

NOTE: This set contains Diethylhexylphthalate (DEHP).


Instructions for Use:

- Close all clamps. Expose outlets of irrigation bags and insert piercing pins. Suspend containers.
- Open pinch clamps. Fill sight chamber to desired fluid level.
- Open roller clamp and fully prime set. Close roller clamp and pinch clamps as required.

- Attach resectoscope to tubing and open roller clamp.
- Add additional irrigation solution as required for the procedure.

CAUTION: Replace per CDC guidelines or health care provider policy.

Made in Costa Rica.
Product inquiries should be directed to
ICU Medical, Inc.,
Lake Forest, Illinois, 60045, USA

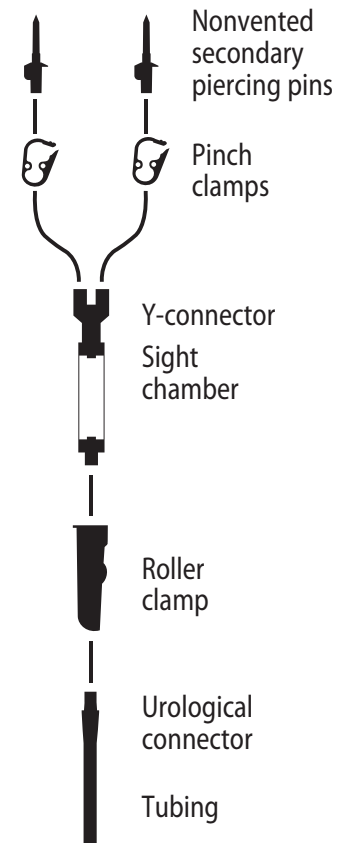

BLP0000014 (09/2019)

PART NUMBER / DOCUMENT NUMBER
BLP0000014 / PLM0001883

This commodity replaces TY-0401

Label copy created using drawing DE015500 and Template TDE015500 (PLM0001899) - FOR LABEL CONTROL USE ONLY.

| | |
|----------------------------|-----------------------------------|
| Adhesive | Perfecseal's CR-90 |
| Material | 6.9 mil DuPont Tyvek, Style 1059B |
| Print Color | PMS Black |
| Dimensions | 253.4 mm x 172.25 mm |
| Bar Code Requirements | |
| Symbology | HIBC LIC Code 128 Subset B & C |
| Narrow Bar Width | 14 mil |
| Bar Width Reduction | 5 mil |
| Bar Height | 0.6 inch |
| Human Readable Information | +M33565430107 |

For additional information and acceptance criteria: See current edition of MASTER-557, MASTER-602, and MASTER-6106.

END OF DOCUMENT