

TDE015400

STERILE R

single  
usedo not  
resterilize

Rx only

List No. 6544-01

# CYSTOSCOPY/IRRIGATION SET

## Nonvented, 77 inch

For Constant or Intermittent Bladder Irrigation  
For Use with Flexible Irrigation Container System

**Natural rubber latex has not been used in the manufacture of the device. Contents sterile and nonpyrogenic in intact unit package. Use aseptic technique. Remove caps when required and secure connections.**

**CAUTION: Not for insertion into blood or plasma containers.**

**NOTE:** This set contains Diethylhexylphthalate (DEHP).

### Instructions for Use:

Close roller clamp. Expose outlet of irrigation bag and insert piercing pin with twisting motion. Suspend container. Squeeze sight chamber to establish desired fluid level. Proceed with Cystoscopy or Irrigation procedures below:

### FOR CYSTOSCOPY PROCEDURE:

- Partially open roller clamp and allow solution to slowly expel air from tubing.
- Close roller clamp and attach distal tubing to cystoscope.

### FOR IRRIGATION PROCEDURE:

- Partially open roller clamp and allow solution to slowly expel air from tubing.
- Remove distal tubing from urological connector.

### For constant or intermittent bladder irrigation:

Attach urological connector to irrigation port of 3-way urinary catheter.

### For high-flow intermittent bladder irrigation:

Use a 2-way Foley catheter attached to a Y-connector. Attach urological connector to one arm of the Y-connector. Connect other arm to drainage set.

- Adjust flow with roller clamp.


**NOTE:** If more than one container is desired, attach with Nonvented Secondary Irrigation Set (List No. 6541, not included).

**CAUTION: Replace per CDC guidelines or health care provider policy.**

Made in Costa Rica.

Product inquiries should be directed to  
ICU Medical, Inc.,  
Lake Forest, Illinois, 60045, USA

BLP0000015 (09/2019)

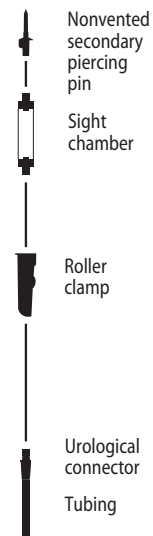

\*\*M33565440108\*

**icu**medical

**PART NUMBER / DOCUMENT NUMBER**  
BLP0000015 / PLM0001884

---

This commodity replaces TY-0402

Label copy created using drawing DE015400 and Template TDE015400 (PLM0001898) - FOR LABEL CONTROL USE ONLY.

Adhesive	Perfecseal's CR-90
Material	6.9 mil DuPont Tyvek, Style 1059B
Print Color	PMS Black
Dimensions	210 mm x 187.5 mm
Bar Code Requirements	
Symbology	HIBC LIC Code 128 Subset B & C
Narrow Bar Width	14 mil
Bar Width Reduction	5 mil
Bar Height	0.6 inch
Human Readable Information	+M33565440108

For additional information and acceptance criteria: See current edition of MASTER-557, MASTER-602, and MASTER-6106.

END OF DOCUMENT