

BORDERED GAUZE

ADHESIVE ISLAND WOUND DRESSING

Bordered Gauze is used as a primary or secondary dressing for partial and full-thickness wounds, including surgical and drain sites.

Indications

Partial and full-thickness wounds including surgical sites.

Directions for Use



1. Remove the Bordered Gauze from the package.



2. Remove one side of the paper liner. Place dressing over the wound and surrounding skin.



3. Remove remaining liner. Smooth dressing over skin and wound.

Removal Instructions

Gently press on the skin and lift an edge of the dressing. Carefully stretch the dressing along the skin to minimize discomfort.

Frequency of Dressing Change

Change the dressing as indicated by the amount of drainage or as frequently as the primary dressing indicates.

General Precautions and Observations

May be used as a water resistant primary or secondary dressing. Should any signs of redness or irritation appear, discontinue use and consult a healthcare professional.

Storage

Protect from freezing. Avoid excessive heat. Sterile in unopened, undamaged package. Single use only.

Contraindications

Third degree burns.



1-800-MEDLINE (1-800-633-5463)

www.medline.com/woundcare

Manufactured in China for Medline Industries, Inc.,

Three Lakes Drive, Northfield, IL 60093 USA. RF18SHZ

N: 06327-510-37



(01)10060196759323

REF MSC3248

4 in x 8 in (10.2 cm x 20.3 cm) DRSG 2 in x 6 in (5.1 cm x 15.2 cm) PAD
1 EACH NOT MADE WITH NATURAL RUBBER LATEX STERILE