ShortStop ADVANTAGE™

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INSTRUCTIONS FOR USE:

DESCRIPTION:

Temporary sharps holder with multiple access points.

INDICATIONS FOR USE: The ShortStop temporary sharps holder with multiple access points is intended to help prevent disease in healthcare workers by reducing the possibility of accidental sharps injury.

CAUTION: Do not reuse.

CAUTION: All needles should be flushed prior to use.

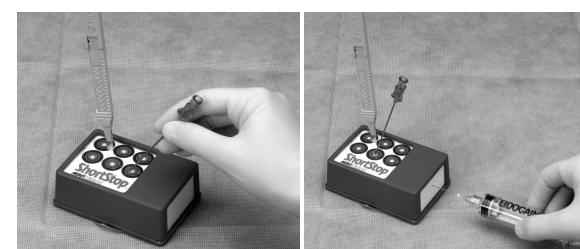
CAUTION: When inserting sharps into the temporary sharps holder, do not support the device with your hands as inadvertent needle sticks may occur.

CAUTION: Needle bevel contact with the plastic wall of the device, may result in damage to the needle.

INSTRUCTIONS FOR USE: Inspect the device prior to use to verify that no damage has occurred during shipping.

1. Place the ShortStop Advantage in the sterile field with the printed foam facing up. This can be accomplished by removing the adhesive thumb tab from the bottom, and pressing firmly to appropriate surface.

2. After use, the clinician may temporarily place a sharp device in either the top access compartment or side access compartment of the temporary sharps holder.

3. After completion of procedure, dispose of the sharp devices using universal precautions. Place the sharp devices in an OSHA compliant waste disposal system. After the sharp devices have been removed from the foam, place the used ShortStop Advantage device in a hospital compliant waste disposal system.

MERIT MEDICAL™

CE 2797

www.merit.com

Manufacturer:
Merit Medical Systems, Inc. South Jordan, Utah 84095
U.S.A. 1-801-253-1600 U.S.A. Customer Service 1-800-356-3748

EC REP Authorized Representative:

Merit Medical Ireland Ltd, Parkmore Business Park West, Galway, Ireland
European Customer Service by Country: Belgium 0800 72906;
France 0800 916030; Germany 0800 1820871; Ireland 091 703700;
Neth. 0800 0228184; U.K. 0800 973115

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