


Product Number: 2006S
Description: Vial Decanter

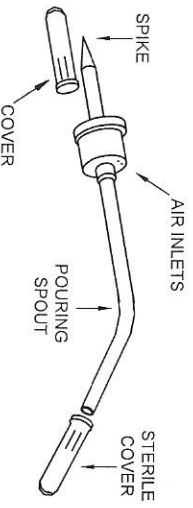


1. Peel back package at arrow. Do not remove vial decanter from package.
2. Hold vial decanter by spout while still in package. Expose spike end of vial decanter and remove protective cap from spike.
3. Spike into rubber stopper with vial standing vertical on flat surface. Push vial decanter down until it is flush to top of vial. Remove peel back pouch.
4. Remove protective cap rapidly from vial decanter to break any airlock. Then proceed to decant medication from vial into container.



Manufactured for:
Microtek Medical, Inc.
602 Lehnberg Road
Columbus, MS 39702 U. S. A.
www.microtekmed.com

Made in China

Product Number: 2006S
Description: Vial Decanter

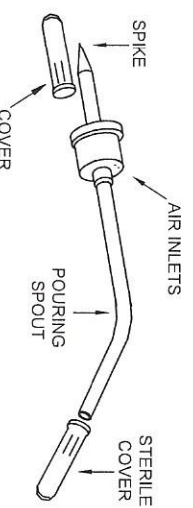

1. Peel back package at arrow. Do not remove vial decanter from package.
2. Hold vial decanter by spout while still in package. Expose spike end of vial decanter and remove protective cap from spike.
3. Spike into rubber stopper with vial standing vertical on flat surface. Push vial decanter down until it is flush to top of vial. Remove peel back pouch.
4. Remove protective cap rapidly from vial decanter to break any airlock. Then proceed to decant medication from vial into container.



Manufactured for:
Microtek Medical, Inc.
602 Lehnberg Road
Columbus, MS 39702 U. S. A.
www.microtekmed.com

Made in China

Do Not Use if Package is Damaged
Do Not Re-use
STERILE EO Sterilized using Ethylene Oxide

Do not Reseal
Do not Resinify
Not made with Natural Rubber Latex

Consult Instructions for use

Rx ONLY
2006S-IFU_2013-08/Rev. C

Do Not Use if Package is Damaged
Do Not Re-use
STERILE EO Sterilized using Ethylene Oxide

Do not Reseal
Do not Resinify
Not made with Natural Rubber Latex

Consult Instructions for use

Rx ONLY
2006S-IFU_2013-08/Rev. C