

# INSTRUCTIONS FOR USE: Anti-fog Solution

## Directions:

Provides an effective method of preventing fog when an endoscope is inserted into the body.

1. Remove paper backing from sponge and place sponge onto sterile drape in a convenient location.
2. Remove cap and apply 5 or 6 drops of Anti-Fog solution to sponge.
3. Touch distal end of scope to sponge, blot dry (do not wipe dry) with sterile sponge.
4. Apply one drop of Anti-fog solution to eye piece of scope, blot dry (do not wipe dry) with sterile sponge.
5. Repeat as needed.

**WARNINGS:** Avoid contact with eyes. If contact with eyes occurs, flush immediately and contact a physician.

**CAUTIONS:** Re-use of this single use device is prohibited by the manufacturer due to risk of cross contamination.


Xodus Medical recommends the use of gamma radiation for sterilization.

LITIFUAF003 R3 5/21/2021



702 Prominence Drive  
Pittsburgh, PA 15068 U.S.A.

Tel: 724.337.5500  
Fax: 724.337.0555

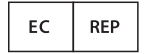

xodusmedical.com  
info@xodusmedical.com

FDA REGISTERED  
ISO 13485 CERTIFIED



EMERGO EUROPE  
Prinsessegracht 20  
2514 AP The Hague  
The Netherlands

