Surgical N95 Particulate Respirator For Use in Healthcare Settings

Check prior to use:

- Check for any visible damage.
- Check expiration date.
- Damaged or soiled particle filtering half respirator or particle filtering half respirator where the expiration date has elapsed must not be used.

Donning Instructions:

1. Open the upper and lower panels of the respirator completely, bending the nose-piece with index finger and thumb at center of the foam to form a curve.
2. Cup the respirator on your face so that the foam rests on your nose and the lower panel is securely under your chin.
3. Pull the top headband over your head and position it high on the back of the head. Then, pull the bottom headband over your head and position it around your neck and below your ears. Adjust for a comfortable fit.
4. Continue to adjust for a comfortable fit by pulling the upper panel toward the bridge of your nose and positioning the lower panel under your chin.
5. Conform the nose piece to your nose by firmly pressing down with your fingers. Inhale and exhale sharply. If air leaks around your nose, readjust the nosepiece. If air leaks between the face and face seal of the respirator, reposition it by adjusting the panels and straps. If you cannot achieve a proper seal, do not enter the contaminated area. See your supervisor.

Fitting Instructions:

- Nosepiece side up / Middle panel facing out.
- Follow donning instructions each time respirator is worn.
- Check facial seal to assure proper fit before entering any contaminated area.
 - » Adjust nosepiece if there is air leakage around the nose.
 - » Reposition headbands to secure facial seal around edges of respirator.
- Do not enter a contaminated area if a proper fit cannot be obtained. Follow all instructions and warnings on the use of this respirator, obtain a proper fit, and wear during all times of exposure.
- Failure to do so will reduce respirator effectiveness and user protection, and may result in sickness or death.

Doffing Instructions:

Warning: The outside of the respirator is considered contaminated!

1. Without touching the outside of the respirator, grasp the **BOTTOM** strap of the respirator and lift it over the head.
2. Grasp the **TOP** strap of the respirator and lift it over the head.
3. Dispose per facility's protocol and wash your hands.

The N95 Respirator (N95SRGRESP-REG) is NIOSH approved as an N95 particulate filter respirator. They provide 95% filtration efficiency of 0.3 micron particles.

Intended Use:

The Surgical N95 Respirator is intended for use in healthcare settings against particulate aerosols free of oil; time use restrictions may apply.

Use Instructions:

1. Before use, the wearer must be trained by the employer in proper respirator use in accordance with applicable safety and health standards for the contaminant and exposure level in the assigned area.
2. If the respirator becomes damaged or breathing becomes difficult, leave the area. Discard and replace the respirator.

⚠ Warning: Before occupational use of this respirator, a written respiratory protection program must be implemented to meet all the local government requirements. In the United States, employers must comply with OSHA 29 CFR 1910.134 which includes medical evaluation, training, and fit testing. Important: Follow all instructions and warnings on the use of this respirator. Wear during all times of exposure. Failure to do so will reduce respirator effectiveness and user protection, and may result in sickness or death.

Time use limitations: If respirator becomes damaged, soiled, or breathing becomes difficult, leave the contaminated area and properly dispose of respirator.

⚠ Warning: 1. Do not wear this respirator when conditions prevent a good facial seal. Reminder: It is important to fit check the respirator each time it is worn. 2. Never alter or modify this product.

Contraindications:

- Respiratory protective devices are not suitable for persons with beards, sideburns or deep scars in the area of the sealing lines of the facepiece.
- In the event of an oxygen deficiency, harmful concentrations of hazardous substances or unknown conditions, self-contained closed-circuit compressed oxygen breathing apparatus must be used independent of the environment.
- Not to be used in areas with open flames.
- The user must be familiar with the use and handling of the device.
- Type and concentration of the hazardous substances must be known.
- Particle filtering half respirators do not offer protection against gases and vapors. The respirator must be replaced when high airway resistance is noticeable.
- The respirator should only be used for maximum one working shift.
- Respiratory protective devices must be selected according to the type and concentration of the hazardous substance.

**PROUDLY MADE IN THE USA
WITH IMPORTED COMPONENTS.**

Resource Optimization & Innovation, LLC
1100 Dr. Martin Luther King Jr. Blvd., Suite 1100
Nashville, TN 37203 • 855-845-9302

Resource Optimization & Innovation, LLC
1100 Dr. Martin Luther King Jr. Blvd., Suite 1100
Nashville, TN 37221 • 855-845-9302

This respirator is approved only in the following configuration:

TC-	Protection ¹	Respirator	Cautions and Limitations ²
		N95SRGRESP-REG	
84A-9745	Surgical/N95	X	ABCJMNOS

1. Protection

N95 - Particulate Filter (95% filter efficiency level).
Effective against particulate aerosols free of oil; time use restrictions may apply.

2. Cautions and limitations

- A. Not for use in atmospheres containing less than 19.5 % oxygen.
- B. Not for use in atmospheres immediately dangerous to life or health.
- C. Do not exceed maximum use concentrations established by regulatory standards.
- J. Failure to properly use and maintain this product could result in injury or death.
- M. All approved respirators shall be selected, fitted, used, and maintained in accordance with MSHA, OSHA, and other applicable regulations.
- N. Never substitute, modify, add, or omit parts. Use only replacement parts in the configuration as specified by the manufacturer.
- O. Refer to user instructions, and/or maintenance manuals for information on use and maintenance of these respirators.
- S. Special or critical user instructions and/or specific use limitations apply. Refer to User Instructions before donning.

S - Special or Critical User Instructions

The N95SRGRESP-REG N95 has been manufactured by HealthPlus, LLC for private label Resource Optimization & Innovation, LLC under TC-84A-9745.

This respirator has been approved as a NIOSH N95 filtering facepiece respirator, for use in healthcare settings, as a Surgical Respirator conforming to recognized standards for Biocompatibility, Flammability Class I, and a Fluid Resistance Level 3 of 160mmHg. Before occupational use of this respirator, a written respiratory protection program must be implemented meeting all the local government requirements. In the United States, employers must comply with OSHA 29 CFR 1910.134 which includes medical evaluation, training, and fit testing.

	Fluid Resistance - Level 3 (160mmHg)		Flammability - Class I (Test 16 CFR Part 1610)
	Do not reuse		Consult Instructions for Use
	Caution		Keep dry and under 85% humidity
	Upper and Lower Temperature Limit		Keep Away from Sunlight and Radioactivity

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