



Instructions:

Surgical N95 Particulate Respirator

Regard's Surgical N95 Particulate Respirator is a three-panel flat fold design used for at least 95 percent filtration efficiency against certain particulates. The unique design is comfortable to wear and fits a broad range of face sizes.

APPLICATION – PROPER FIT FOR MAXIMUM PROTECTION



1 Prepare: Remove from packaging and hold with foam for nose on top side.



2 Open: Unfold top and bottom panels fully & bend the nosepiece over your thumb at the foam's center. Ensure the bottom panel is unfolded & completely open.



3 Position: Place the respirator on your face with the foam on the bridge of your nose and the bottom panel securely under your chin.



4 Secure Straps: Pull the top strap to the **crown of your head and place it above your ears**. Pull the bottom strap to the **nape of your neck**, positioned below your ears. Do not cross them.



5 Final Fit: Adjust panels for comfort and a snug fit, pulling the top panel toward the bridge of your nose and the bottom panel under your chin. Ensure no hair, jewelry, or clothing is between your face and the respirator seal.



6 Mold Nosepiece: Using **both hands**, push inward while moving fingertips down both sides of the metal nosepiece to mold it to your nose shape. Always use two hands; **do not pinch** with one hand, as this may result in improper fit and allow for leaks.



PERFORM A USER SEAL CHECK Always check your seal before use.

Place one or both hands completely over the middle panel. Inhale and exhale sharply. Be careful not to disturb the position of the respirator. If air leaks around your nose, re-adjust the nosepiece as needed. If air leaks around respirator edges, adjust panels and position of straps and make certain respirator edges fit snugly against the face. **If you cannot achieve a proper seal, do not enter the contaminated area. See your supervisor.**

REMOVAL



1 Without touching the respirator facepiece, slowly lift the bottom strap around your neck and up over your head.



2 Lift off the top strap. Do not touch the respirator.



3 Store or discard according to your facility's infection control policy and wash your hands.

Warning: Respirators help protect against certain airborne contaminants. Before use, the wearer must read and understand the User Instructions provided as a part of the product packaging. Follow all local regulations. In the U.S., a written respiratory protection program must be implemented meeting all the requirements of OSHA 29 CFR 1910.134 including training, fit testing and medical evaluation. **Misuse may result in sickness or death.** For correct use, consult supervisor and User Instructions.

ORDERING & SERVICE INFO
support@roiscs.com • 1-855-845-9302
Specifications subject to change without prior notice.

IMPORTANT ORDERING INSTRUCTIONS
Orders must be placed directly with ROi using ROi reorder numbers to receive HealthTrust contracted pricing. Specifications subject to change without prior notice.

ROi Distributed by
Resource Optimization
& Innovation, LLC
Nashville, TN 37203